Tanım

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CE 2195

Descriptions

Surgical instruments designed for use with the Nucleoss Dental Implant System manufactured by Sanililar Tibbi Cihazlar Med. Kim. San. Tic. Ltd. Şti. are intended for transient use in the oral cavity (no more than 60 minutes at a time).

Indications

Surgical instruments are used to prepare implant sites, placement in the bone of the upper or lower jaw for Nucleoss Dental Implants, to insert the implants in the sites, to tighten and unscrew all the fixation screws.

Contraindications

When assessing the patient, in addition to his/her eligibility as regards implant-restoration rehabilitation, it is usually necessary to consider the contraindications that apply to oral surgery procedures in general. These include: Clotting disorders, anticoagulant therapy. Healing or bone regeneration disorders such as, for example: decompensated diabetes mellitus, metabolic or systemic diseases that compromise tissue regeneration with a particular influence on healing and bone regeneration, alcohol abuse, smoking and use of drugs. Immunosuppressive therapy, such as: chemotherapy and radiotherapy. Infections and inflammations, such as: periodontitis, gingivitis. Poor oral hygiene. Inadequate motivation. Occlusion and/or articulation disorders as well as an inadequate interocclusal space. Inadequate alveolar process.

It is contraindicated to fit implants and implant restorations in patients with poor general or oral health, those who are unable to monitor their general conditions properly or those who have had organ transplants. Psychologically unstable patients, alcohol or drug abusers, and poorly motivated or uncooperative patients should also be considered unsuitable for this kind of treatment. Patients with poor periodontal health should first be treated and allowed to recover. In the presence of a lack of bone substance or poor quality of the receiving bone, such as to compromise the stability of the implant, suitable guided tissue regeneration must be performed prior to implant treatment. Contraindications also include: allergy to titanium, acute or chronic infective diseases, sub-acute chronic maxillary osteitis, systemic diseases, endocrine disorders, diseases resulting in microvascular disorders, pregnancy, breastfeeding, previous exposure to radiation,

Procedures

For detailed information please consult the Surgical Manual.

Precautions

Surgical instruments used to achieve and maintain osseointegration should be utilized by persons trained in this method. Pre-operative hard tissue or soft tissue deficits may result in a compromised esthetic outcome or unfavorable implant angulation. Pre-operative patient evaluation and close cooperation between surgeon, restorative dentist and dental laboratory technician is essential for success.

Sterility

Nucleoss surgical instruments are sold in NON-STERILE packs. Before use, they must be cleaned, disinfected and sterilised according to the instructions reported below. Failure to follow these warnings may expose the patient to infection.

Cleaning/Disinfection and Sterilization for NucleOSS Reusable Surgical Instruments

Point of use

To avoid contamination of the kit box (mainly the instrument plate) and damage of the sterile gloves by the sharp drills, it is recommended to pick up the devices using a pair of tweezers. Immediately after surgery, rinse the devices with running water to remove all gross visible blood, body fluids and tissue. If cleaning is likely to be delayed, immerse the devices in a bath of lukewarm cleaning solution to avoid drying of soil.

Transport

Avoid mechanical damage, e.g., do not mix heavy devices with delicate ones. Pay particular attention to cutting edges, both to avoid injury and to avoid damage to the medical devices. Get the medical devices to the point where cleaning is to be performed as soon as practical. If transfer to the processing area is likely to be delayed, consider covering the medical devices with a damp cloth or store the medical devices in closed boxes to avoid drying of soil and/or debris. When longer delays are expected, immerse the devices in a bath of a lukewarm cleaning solution to avoid drying of soil

Preparation and disassembly

Kit boxes must be disassembled completely before cleaning and disinfection.

Manual Cleaning, Disinfection and Drying

(incl. Pre-cleaning)
Pre-cleaning
Immediately after surgery, rinse the devices with cold running water to remove all gross visible blood, body fluids and tissue. If cleaning is likely to be delayed, immerse the devices in a bath of lukewarm cleaning solution to avoid drying of soil. Scrub the outer of the instruments with a soft bristled nylon brush under cold tap water until all visible soil and debris is removed.

Check the channels for residual soil and/or debris. Immerse the instruments in the ultrasonic bath (at least 35 kHz) with demineralised water with a cleaning additive (1.0 % Neodisher MediZym, 30 °C) for 5 minutes
The final rinsing of the instruments must be done with cold deionized water for 1 minute.

Manual Cleaning

The instruments must be placed completely in a cleaning bath (0.5 % Neodisher MediZym solution in tap water according to the manufacturer's instructions for the cleaning agent, at 30°C±3°C) for 20 minutes, and they must not touch each other.

Flush the internal channels with cleaning solution using the irrigation needle connected to a syringe. Check the channels for residual soil and/or debris.

After the exposure time, scrub the outer of the instruments, especially hard-to-reach areas for one minute with a soft bristled nylon brush under cold tap water until all visible soil and debris is removed.

Rinse for at least 1 minute with freshly prepared purified water / highly purified water or sterile water until all traces of cleaning solution are removed.

Manual Disinfection

Prepare a bath with Cidex Opa at the minimum 0.3% concentration and 20°C/68°F temperature specified in the detergent manufacturer's instructions and immerse the devices for at least 12 minutes.

Flush the internal channels a minimum of three times with disinfection solution using the irrigation needle connected to a syringe.

Thoroughly rinse the instruments for a minimum of 1 minute with freshly prepared purified water or sterile water. Repeat this stage another two times, for a total of three rinses. Do not reuse the water. Always use fresh volumes of water for each rinse.

Flush the internal channels/lumina a minimum of three times with freshly prepared purified water or sterile water using the irrigation needle connected to a syringe.
Dry the instruments using medical compressed air and clean and lint-free single-use wipes.

Aletleri ıbbi basınçlı hava ve temiz ve tıy bırakmayan tek kullanımlık mendiller kullanılarak kurutun.
Before preparing for sterilization, check all instruments after cleaning/disinfection for visible soil, corrosion, damaged surfaces and contamination and sort out damaged instruments.

Assembly and Packaging of Kit Boxes

Where appropriate, the cleaned, disinfected, and checked medical devices should be assembled.
Complete kit boxes with assembled instruments must be sterilized in their 'ready for use' state. There is no need to disassemble these instruments for sterilization.
To sterilize single components and instruments, place them in sterilization pouches.
Place the kit box in a sterilization pouch to protect from contact with contaminated instruments until sterilization by autoclave.

Sterilization

Recommended sterilization condition as follows;

Method	Moist heat sterilization according to ISO 17665
Cycle	Pre vacuum
Temperature	132°C / 270°F
Exposure time	4 minutes
Pre vacuum	3 times < 60 mbar
Minimum drying time	20 minutes in chamber

Storage

After sterilization, place the devices in a dry and dark place such as a closed cupboard or drawer.

Note1: Recommended Cleaning and Disinfecting Agents

Cleaning and disinfection step	Detergents used in Nucleoss validation
Pre-cleaning	1.0 % Neodisher MediZym, 30°C for 5 minutes.
Manual Cleaning	0.5 % Neodisher MediZym, at 30°C±3°C for 20 minutes
Manual Disinfection	Cidex Opa Minimum effective concentration 0.3% Minimum temperature 20°C/68°F Immersion time at least 12 minutes

Note2: For cleaning/disinfection only specifically formulated cleaning/disinfection agents should be used. For time and concentration, detergent manufacturers' instructions should be observed.

When selecting the cleaning agents and disinfectants please make sure that they do not contain the following ingredients: organic, mineral and oxidizing acids (minimum permissible pH value 5), strong alkalis (maximum permissible pH value 9, mildly alkaline cleaners recommended), organic solvents (e.g. alcohols, ethers, ketones, benzenes), oxidation agents (e.g. hydrogen peroxide), halogens (chlorine, iodine, bromine), aromatic/halogenized hydrocarbons, salts of heavy metals,

Note3: Water quality; The quality of the water used for diluting cleaning agents and/or disinfectants and for rinsing medical devices should be carefully considered. Application of freshly prepared purified water / highly purified water or sterile water for rinsing purposes (according to the pharmacopoeias) with less than 10 cfu/ml and 0.25 EU/ml is highly recommended. Mineral residues from hard water as well as higher contamination with microorganisms and endotoxins can result in staining of the device or prevent effective cleaning and disinfection.

Reusability

Frequent processing has minor effects on the instruments. The end of product life is normally determined by wear and damage during use (cutting instruments are an exception; see below). Therefore, instruments can be reused with appropriate care, provided they are undamaged and not contaminated. Do not use instruments beyond the effective product life cycle nor use damaged and/or contaminated instruments.

Cutting Instruments

If appropriately cared for, and provided they are undamaged and not contaminated, the cutting instruments can be reused up to maximum of 15 times (1 time use = placement of 1 implant); any further use extending beyond this number of use

REF Catalog Number

LOT Lot Number

Caution

Consult Operating Instructions

Manufacturer

Do Not Use if Package is Damaged

Non-Sterile