Prof. Dr. Oğuz Ozan
Prof. Dr. Oğuz Ozan

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Oguz Ozan who was born in 1978, graduated from Ankara University Faculty of Dentistry. He received the PhD degree from the same university, of Prosthodontics where his doctoral research was named "Evaluation of Planning Success of Implant Supported Prosthesis Guided with a 3 Dimensional CT Based Computer Software". His works has been published in major international journals extensively. He is actively involved as a director in multiple collaborative research projects. He has given several international lectures and has awarded multiple times. He is the general secretary of Computer Aided Implantology Academy and Together for Implantology board member. He got Assoc. Prof. in 2012, and titles as professor in 2017. He is still giving courses and practising in his private clinic.

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*Patient Information
The patient was born 12.03.1956 and a non-smoker female.
Patient History:
There is no systemic disease.*

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Panoramic radiograph of the patient who is consulted to the clinic for the treatment of the mandibular teeth.

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The patient who was carrying two removable denture on upper teeth consulted to the clinic for the loose lower teeth treatment by fixed restoration.

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The patient was diagnosed with periodontitis, lost the bone support completely and the whole teeth was extracted. 6 Nucleoss T6 implants are placed and multiunit abutments are connected with implants during surgery. Immediate temporary prosthesis is planned, so no hex titanium caps are used.

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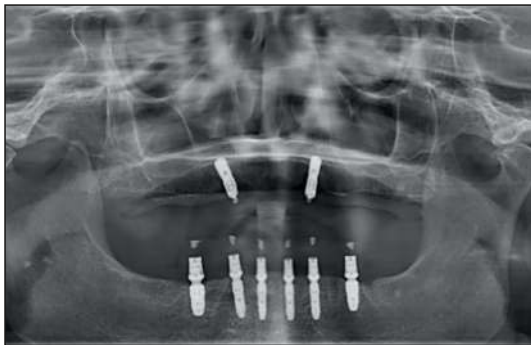
At the end of the 2 months osseointegration period, the temporary prosthesis has removed. The temporary prosthesis that has applied by following the hygiene procedures caused minimum infectious areas. After this stage, the treatment has completed with screw retained one piece fixed restoracion

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The temporary acrylic prosthesis that is finished similarly with the patient's previous upper prosthesis. Titanium caps are connected rigidly and the next day it is connected by multiunit abutments.

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The post operation x-ray

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Final figure of the prosthesis

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Intraoral final figure of the prosthesis

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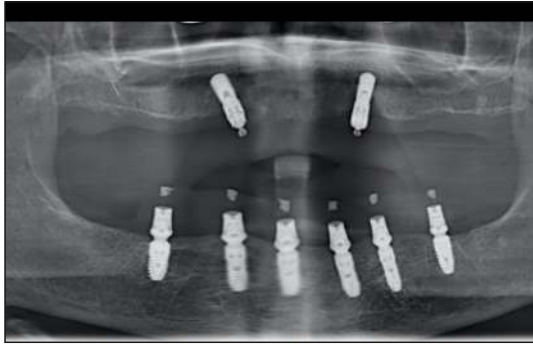
25 months after figure of the prosthesis

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Intraoral figure of the prosthesis after 25 months

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