M.Sc.Dr. Khagani Huseynov

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He graduated from Azerbaijan Medical University Faculty of Dentistry in 2001. In 2006, he completed the expertise course on Maxillofacial Surgery and Implantology in Post-Diploma Training Academy, St. Petersburg University. In 2011, he received his master degree in laser dentistry at Achen University, Germany. In 2012, he completed the surgical courses of Prof. Dr. Ateş Parlar. In 2015 he graduated from the education program of Implantology in Frankfurt am Main University, Germany. He is also the head of the Azerbaijan Esthetics Dentistry Academic Association). He has been various organising congresses and giving seminars and courses in his country since 2008.

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Patient Information

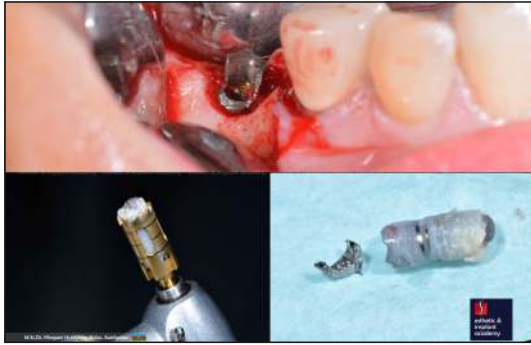
57 years old, with good mouth hygiene, non-smoker female.

Patient History: The patient consulted to the clinician with a slight pain during chewing and inbalanced crown upon the implant. As a result of the examination the broken implant has detected.

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
Mounting small diametered implant (D3,3) has resulted with neck fracture after years. Implant will be removed and the new implant will be planned.

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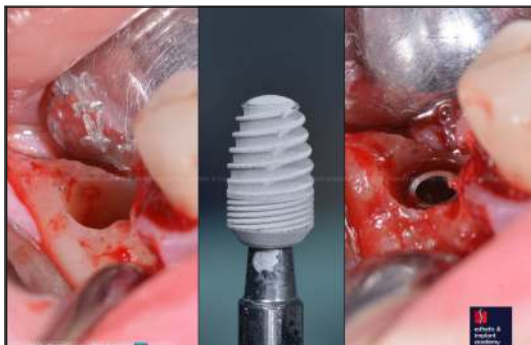
Implant has removed with trephine bur. Titanium particles has cleaned away with well washing.

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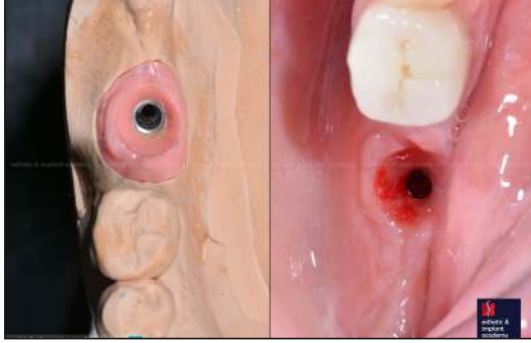A Nucleoss T6 D4. 8/L8 mm implant has mounted. In accordance with the protocol, implant has embedded to the bone by 1 mm depth. The scar has closed by applying standard suturing.

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
After three months, the gingiva former has placed with a minor surgery and the patient was left for recovery for a month. Single tray impression technique has used for measurement. The image of emergence profile and transferring to plaster model.

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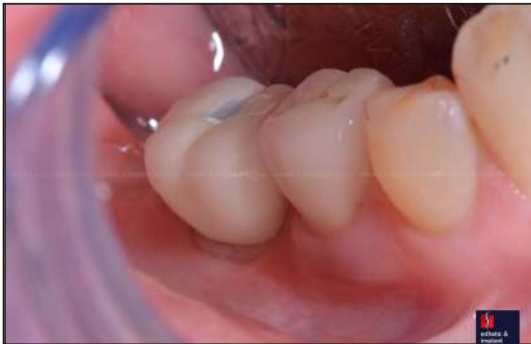
Metal-ceramic crown implemented, extra-oral abutment cementation prepared and attached to the implant with 25 N/cm torque.

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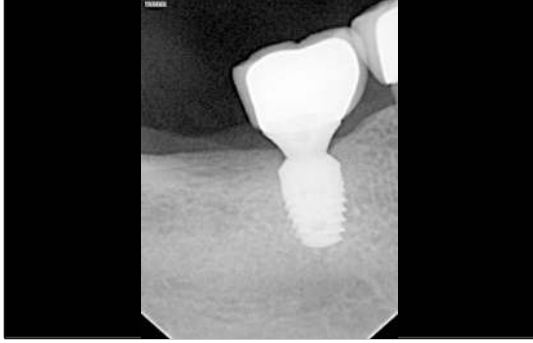Crown and gingiva adaptation is very well. Occlusion has fixed for the comfort of the patient.

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The x-ray after 2 years 8 months. There is no sign of osteolysis, on the contrary, osteogenesis increase observed in the implant circuit.