Prof. Dr. Oğuz Ozan  
Prof. Dr. Oğuz Ozan

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*Oguz Ozan who was born in 1978, graduated from Ankara University Faculty of Dentistry. He received the PhD degree from the same university, of Prosthodontics where his doctoral research was named "Evaluation of Planning Success of Implant Supported Prosthesis Guided with a 3 Dimensional CT Based Computer Software". His works has been published in major international journals extensively. He is actively involved as a director in multiple collaborative research projects. He has given several international lectures and has awarded multiple times. He is the general secretary of Computer Aided Implantology Academy and Together for Implantology board member. He got Assoc. Prof. in 2012, and titled as professor in 2017. He is still giving courses and practising in his private clinic.*

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#### Patient Information

*The patient is a non-smoker female and 55 years old. There is no systemic disease.*

#### Patient History:

*The patient consulted to the clinician with the complaint of missing 34th, 35th, 36th and 37th teeth in the lower left jaw.*

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*Assoc. Prof. Dr. Ismail Doruk Kocyigit graduated from Ankara University Dentistry Faculty in 2002 and in the same year he started his residency training in Oral and Maxillofacial Surgery at the same faculty. He has been assigned Assoc. Prof. Dr. at Kirikkale University, Department of Oral and Maxillofacial Surgery in 2014 and he is currently at the same position in the same university. He is a member of (IAOMS), (EACMS), (ACBID), (CAI Academy). He was joined an observer ship programme in London in Guys' &St.ThomasHospital Cleft Service and The Royal Surrey County Hospital in 2012 with the support of ACBID Scholarship. His principle areas of clinical practice and research are and Reconstruction, Orthognatic Surgery, Cleft Lip and Palate, Trauma&Dental Implants. Dr. Kocyigit has 32 published/ahead of publishing papers in SCI/SCI-expanded journals and severals in national journals and 5 International Scientific Awards. He has been several national and international lectures about his interested topics.*

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### Patient Information

*The patient is a non-smoker male and 44 years old. There is no systemic disease.*

### Patient History:

*The patient has lost the whole teeth caused by periodontal and deep cavities. For this reason fixed prothetic treatment is decided.*

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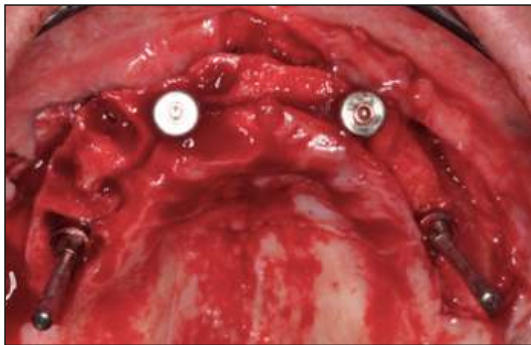
*As understood from the clinical diagnosis of the patient when he consulted to clinic, prosthetic treatment was not possible. Intra-oral view shows that all of the hopeless teeth should be removed due to terminal dentition.*

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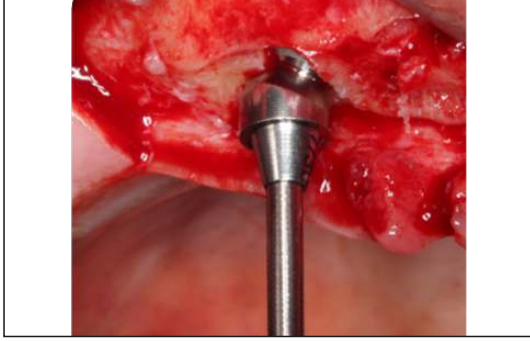
*After the radiographic examination of the patient, extraction of the all teeth and immediate implantation planned. During the treatment plan, 4 implants for the maxilla and 6 implants for the mandibula is planned.*

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*After following the fix on 4 protocol, occlusal view of the upper jaw*

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*30° angled multi-unit and implant fixed during surgery to avoid placement difficulties.*

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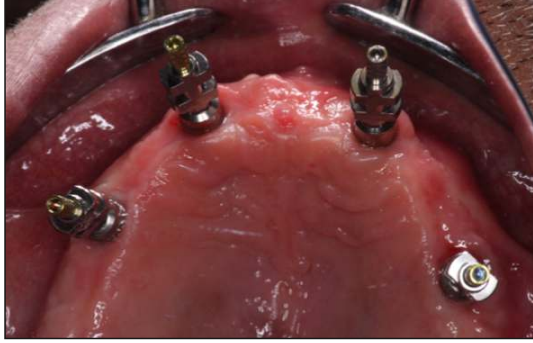
*In the same day, immediat implant loading with provisional fixed prosthesis. Occlusal view of the screws.*

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*Three months of healing period has passed before final prosthesis.*

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*The 3 months after, view of the upper jaw. Impression taking by open tray technique for fixed prosthesis.*

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*During impression and transfer to the stone model, in order to prevent dimensional deviations, open impression tray parts are bended passively, and attached with an orthodontic wire and acrylic.*

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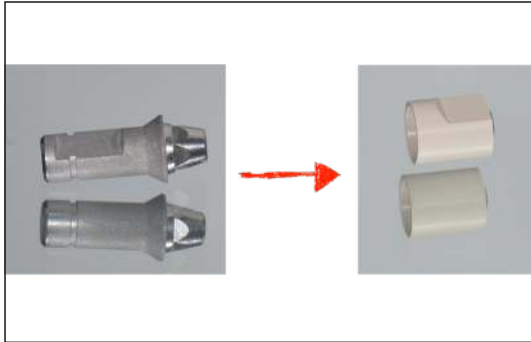
*Stone models of the lower(on the left) and upper (on the right) jaws.*

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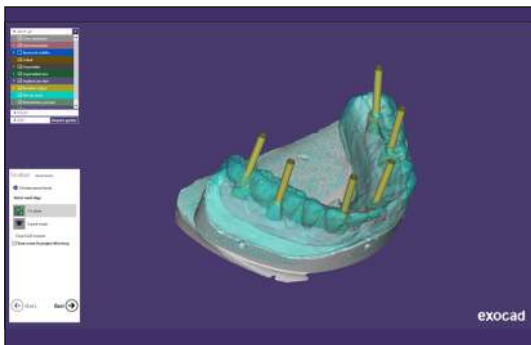
*After the patient's approval of the esthetics that has been shown before, try-on models are sprayed in order to work in the scanner.*

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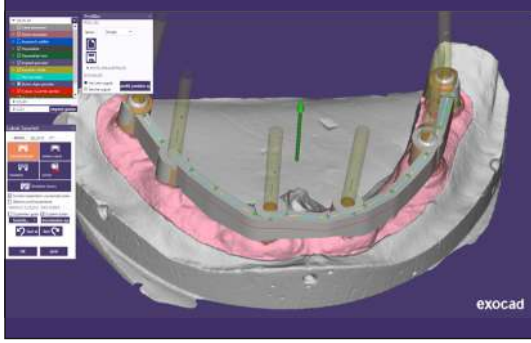
Nucleoss T6 multi unit analogs and scan bodies

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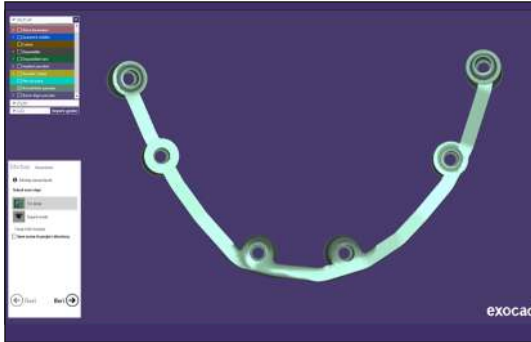
*Scanned lower jaw model and implant positions after matching the software.*

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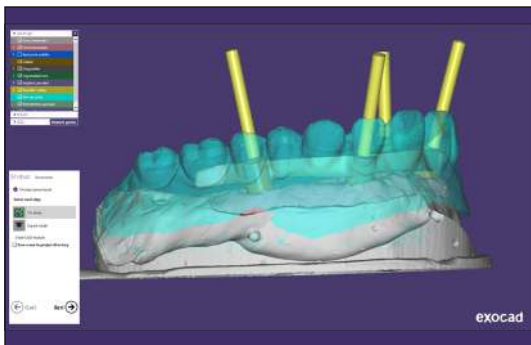
*The titanium suprastructure suitable to scanned lower jaw try-on designed by software*

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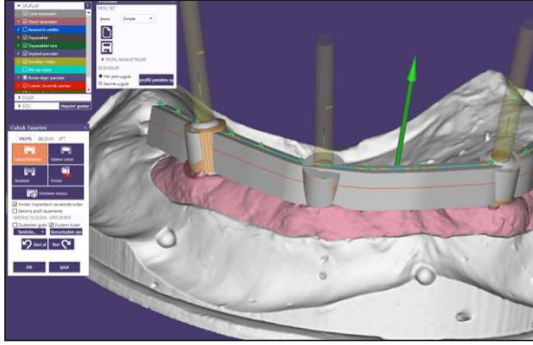
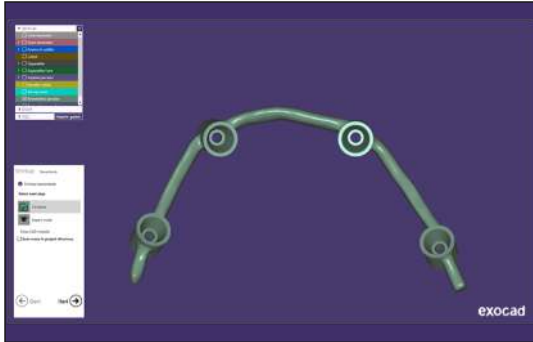
*Scanned upper jaw model and implant positions after the matching the software.*

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*The titanium suprastructure suitable to scanned upper jaw try-on designed by software*

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*Cad/Cam and milled titanium suprastructure for the upper and lower jaw that is prepared by software.*

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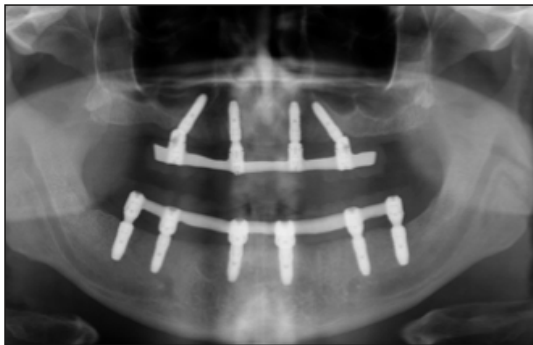
*The view of the titanium based full acrylic prosthesis.*

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*Final prosthesis and the smile line*

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*Follow-up radiography, 44 months after the implant placement*